NITRO

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Table of contents

Preface	10
Declaration of conformity	10
Service life	10
Disposal	10
Identification plate	10
Reuse.....	10
Warranty	10
Parts of the product	11
Indication.....	12
Scope of delivery	12
Safety instructions.....	12
Unfolding.....	12
Assembly of accessories Nitro ≠ Carbon	12
Assembly of accessories Nitro Carbon	13
Height adjustment of push handles.....	13
Adjusting the brakes	13
Braking Parking ≠ Twist brake	13
Braking Parking = Twist brake	13
Sitting	14
Folding Transporting Stowing	14
Backrest adjustment	14
Cleaning Maintenance.....	14
Technical data	15

Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

Declaration of conformity

For the product described, DRIVE MEDICAL GMBH & Co. KG declares compliance according to the medical device regulation (EU) 2017/745.

Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.


This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

Identification plate

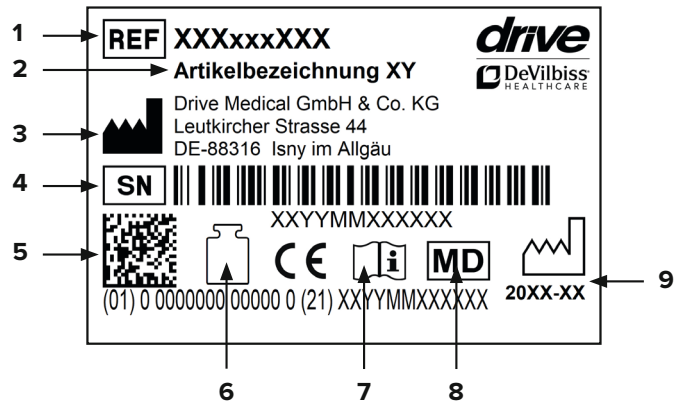1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year-month)

Reuse

This product can be reused.

Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available from your specialist dealer or at www.drivedevilbiss.de

Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

EXCLUSION OF WARRANTY

Parts subject to wear include (among others): wheels, brakes and handles

Parts of the product

NITRO

- | | | |
|-----------------------------------|--------------------------|---------------------|
| 1. Handle | 6. Cane holder | 11. Front reflector |
| 2. Brake | 7. Frame | 12. Bag |
| 3. Rear reflector in handle | 8. Rear wheel | 13. Seat |
| 4. Push handle | 9. Brake adjusting screw | 14. Back support |
| 5. Locking button for push handle | 10. Front wheel | 15. Twist brake |

Indication

INDICATION:

Walking aid for outdoor and indoor use to compensate for or improve limited walking ability.

Insured parties are to be provided with walking aids with the intention of compensating for or at least improving the user's limited walking ability (expansion of radius of activity or mobility).

Persons with limitations of hand/arm functions require particular adjustments/designs of handles or handrests/armrests.

Movable walking aids may not be suitable for persons with balance disorders; in individual cases, this should be tested out. Where appropriate, an attendant must be present.

The rollator is to be used only for normal, everyday activities.

CONTRAINDICATION:

The rollator is not suitable for use in the case of: perception disorders, severe loss of balance, inability to sit down

Scope of delivery

1 x	Rollator
1 x	Bag
1 x	Back support
1 x	Cane holder
1 x	Instructions for use

Safety instructions

- Use the rollator only as a walking aid!
- Use the rollator only on an even and solid surface.
- Rollator may not be used as a seat in vehicles, such as a bus or train!
- It is prohibited to stand on the rollator or to transport a person or animal on the rollator.
- Take note of the permitted maximum weight of the user. This information can be found in the technical data in these instructions for use as well as on the identification plate.
- Use the bag only to transport objects and do not overload it! Max. load: 5.0 kg
- Use on escalators and the like is expressly prohibited!
- Do not hang any bags on the handle – risk of tipping!
- On steep surfaces, use the service brake (pull the brake lever upwards).

- Leaning excessively on the handles can cause the rollator to tip backwards.
- When folding/unfolding the rollator, clothing and body parts can be pinched.
- Protect the rollator from direct sunlight. Product parts can become hot and cause skin injuries if touched.
- Prior to each use, ensure that your rollator is intact and all adjustable parts are properly secured. Check that the brakes are working properly!
- Before using the rollator, ensure that the folding guard is properly locked.
- Check the screws on the rollator regularly and tighten, if necessary.
- The braking effect can be impaired by wear and tear on the tires. For this reason, check the braking effect and tires regularly.
- The rollator should NOT be carried by the seat, removable parts or the brake cables.
- Every serious incident which occurs in connection with the product must be reported to the manufacturer and the competent authority of the member state in which the user and/or patient is located.

Unfolding

Place rollator on its wheels. Now press both side rails of the seat section downwards until they click into place. The rollator is unfolded correctly when the locking hook has clicked into place (**pic. 1**).

Assembly of accessories Nitro ≠ Carbon

Fold the rollator slightly and then insert the bag with the sliding blocks into the guides on the left and right. Unfold the rollator (**pic. 2**).

Assembly of accessories Nitro Carbon

Fold the rollator slightly and then secure the bag to the bars using the Velcro fasteners. Unfold the rollator (**pic. 3**).

Height adjustment of push handles


Press the bottom of the locking button to adjust the height of the push handle while pulling upwards at the same time.

Once the handle moves, release the button. Then this automatically clicks into place at the next possible height.

Repeat this procedure until you have reached the desired height.

Adjusting the brakes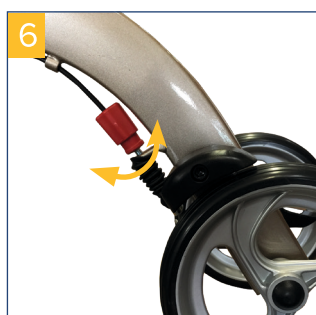

If braking is poor or too hard, the brake can be adjusted using the adjusting screw on the Bowden cable below to achieve the desired braking effect. To do this, press the adjusting sleeve in the direction of the wheel (**pic. 5**) and then rotate the adjusting screw accordingly (**pic. 6**).

Rotation to the right: Brake looser

Rotation to the left: Brake tighter

Braking | Parking ≠ Twist brake

To brake when walking with the rollator, pull both brake handles upwards with your hands (**pic. 7**). Always brake slowly while applying equal pressure with both hands.

To apply the brakes/park the rollator, press both brake handles downwards until they audibly click into place (**pic. 8**).

To release the brake, pull both brake handles upwards once again until they are released (brake handles are in the middle position).

The braking effect can be impaired by wear and tear on the tires. For this reason, regularly check the braking effect.

Braking | Parking = Twist brake

To brake when walking with the rollator, rotate both brake handles inwards and downwards with your hands (**pic. 9**). Always brake slowly while applying equal pressure with both hands.

To apply the brakes/park the rollator, rotate both brake handles inwards and downwards until you feel them click into place (**pic. 10**).

To release the brakes, rotate both brake handles upwards once again until they are released.

The braking effect can be impaired by wear and tear on the tires. For this reason, regularly check the braking effect.

Sitting

Place the rollator on an even and solid surface. Then apply the brakes. The rollator must be stable and may not slip. Only sit upright; do not lean backwards. Seated position permitted only as shown and against the direction of movement (**pic. 11**).

Folding | Transporting | Stowing


To fold up the rollator, use the handle to pull the seat section upwards (**pic. 13**).

Then push the sides of the rollator together until the folding guard audibly clicks into place (**pic. 14**).

Backrest adjustment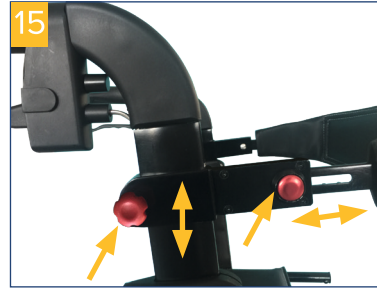

Cleaning | Maintenance

- Clean regularly with a moist cloth!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillol AF.
- Check the screws regularly and tighten, if necessary.
- The product should not be exposed to temperatures above 60°C for long periods of time (for example, use in a sauna, etc.).
- In the event of defects, contact your specialist dealer from whom you purchased this product.
- Check the condition of the tires regularly. They are „fail-safe“, meaning that you do not have to check air pressure or fill them with air. Always check the tires for tears or other damage, however.

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Drive Medical GmbH & Co. KG