

**PHILIPS**

RESPIRONICS

*Wisp*

Minimal contact nasal mask

**OXYCARE** Medical Group GmbH  
FON +49(0)421 48 99 66 • www.oxycore.eu

MEMBER OF  
**SOS** oxygene

Performance has a  
**new style**

# Wisp innovation

## Efficient and simple

Developing solutions that meet the needs of a rapidly changing industry calls for fresh thinking and collaboration. At Philips Respironics, we have traveled around the world asking clinicians, homecare providers, and patients how we could improve their experience.

Clinicians wanted us to resolve the tradeoff between nasal and pillows masks. Homecare providers wanted a solution that could reduce operational complexity by getting more patients served from one package. And patients wanted a better overall mask experience that improved their lifestyle.

When you put it all together, the answer is Wisp.

### Traditional nasal mask

- Larger footprint
- Intimidation factor

- + Seal/stability
- + Comfort

## Wisp

- + Minimal size
- + Lightweight

- Pressure restrictions
- Nasal discomfort

### Traditional pillows mask





# Magnetic personality

## Magnetic clip accessory for enhanced ease of use

The easy-to-use Wisp magnetic clip is standard in the Wisp fitpack and an accessory part that can be retrofitted to existing Wisp masks. It can replace the standard headgear clip to enhance ease of use for patients with limited dexterity.





# Wisp features

Self-adjusting top strap  
with tube holder loop

---


Reversible fabric frame of soft  
suedette and silky sateen (also  
available in a clear silicone)

---

Magnetic clips for  
easy adjustment

---



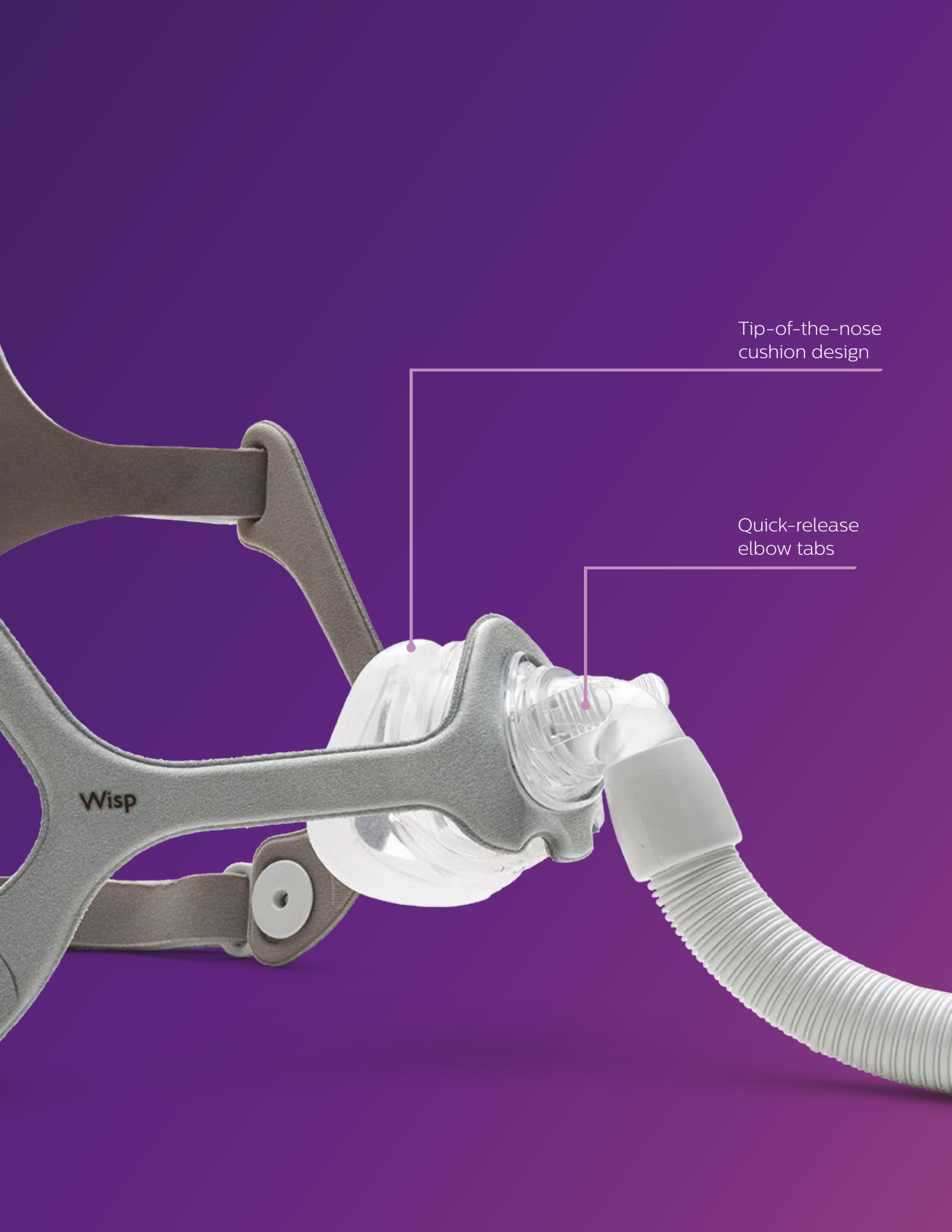

Tip-of-the-nose  
cushion design

Quick-release  
elbow tabs

Wisp

# Wisp patient fitting guide

Follow the instructions to get started with fitting the Wisp nasal mask.

- 

Before putting on the mask, select the cushion that fits the width of the nose without blocking the nostrils. The top of the cushion may sit between the bridge and tip of the nose.

**Note:** Use the mask sizing gauge to select the cushion that fits the width of the nose without blocking the nostrils.
- 

Disconnect the headgear tab or clip. Peel the headgear tabs away from the headgear strap and pull them through the slot in the mask frame, or if using the clip, twist outward.
- 

Pull the headgear over the head and place the cushion over the nose.

**Note:** The headgear should lie flat against the back of the head.
- 

Thread the tab through the slot in the mask frame or press the clip against the headgear slot to reattach.

**Note:** If the clips are not used, the headgear tabs can be threaded through the slots on the mask frame.
- 

Peel the headgear tabs away from the headgear straps and then press them back together to reattach. Connect the side straps to the headgear tab located at the base of the crown strap. Connect the bottom strap tabs near the back of the headgear. Adjust the side and bottom straps evenly.

**Note:** Do not over-tighten. The headgear should feel loose and comfortable.
- 

Connect the Wisp tubing and elbow swivel to the front of the mask. Optionally, slide the mask tubing through the headgear crown loop before pulling on the headgear.
- 

Assume different sleeping positions. Move around until comfortable. If there are any excessive air leaks, make final adjustments while lying down.

**Note:** Using the headgear crown loop can help make some sleeping positions much more comfortable.



# Choose which Wisp works

Wisp features a unique, compact design that is created for maximum comfort. The “**tip-of-the-nose**” cushion is designed to sit below the nose bridge, making minimal contact with the face while creating an optimal seal. Wisp’s sleek design eliminates the need for a forehead pad, offering a better field of vision.



Petite



Small/medium



Large



Extra large

We’ve taken the guesswork out of finding the fit that’s right. The fitpack includes small/medium, large, and extra large sizes, with the petite cushion being sold separately.

## Choice of frame




### Clear frame

The silicone frame is lightweight and flexible, offering a smooth feel for patients who prefer the look of clear silicone.

### Fabric frame

The reversible fabric frame of soft suedette and silky sateen allows patients to choose which side feels best.

## DreamMapper

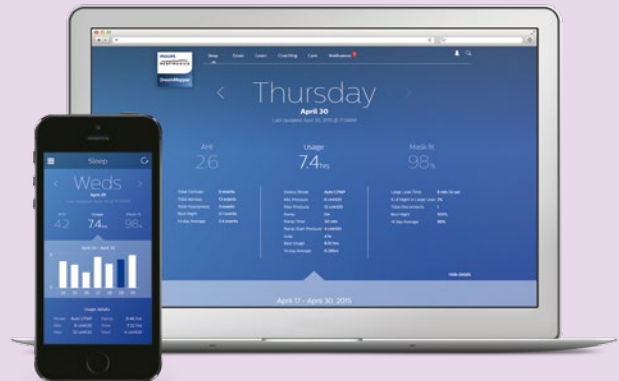

### DreamMapper is the #1 downloaded PAP therapy engagement app<sup>1</sup>

When it comes to your patients reaching their sleep therapy goals, DreamMapper<sup>2</sup> is there for them with the tools, knowledge and resources they can use to stay motivated.

Visit: [www.philips.com/mydreammapper](http://www.philips.com/mydreammapper) to experience DreamMapper.

<sup>1</sup>Source GooglePlay, 5/9/2016.

<sup>2</sup>To see which therapy devices are compatible with DreamMapper, visit: [www.dreammapper.com/compatible](http://www.dreammapper.com/compatible).

# Ordering information

## Wisp fitpack with headgear; small/medium, large, extra large cushions

Wisp mask, clear frame, headgear, magnetic clips	1094050
Wisp mask, fabric frame, headgear, magnetic clips	1094051

## Wisp fitpack without headgear; small/medium, large, extra large cushions

Wisp mask, clear frame, without headgear	1094055
Wisp mask, fabric frame, without headgear	1094056

## Wisp mask, clear frame without headgear

Part number	Part number
Petite	1101550
Small/medium	1101551
Large	1101552
Extra large	1113957

## Wisp mask, fabric frame without headgear

Part number	Part number
Petite	1101553
Small/medium	1101554
Large	1101555
Extra large	1113958

## Wisp mask, clear frame with headgear

Part number	Part number
Petite	1118060
Small/medium	1118061
Large	1118062
Extra large	1118063

## Wisp mask, fabric frame with headgear

Part number	Part number
Petite	1118064
Small/medium	1118065
Large	1118066
Extra large	1118067

## Wisp youth fitpack with reduced-size headgear; small/medium, large, extra large cushions

Wisp mask, fabric frame, headgear, magnetic clips	1109298
---	---------

## Replacement parts

Part number	Part number
Petite Wisp cushion	1094086
Small/medium Wisp cushion	1094087
Large Wisp cushion	1094088
Extra large Wisp cushion	1112031



## Replacement parts

Part number	Part number
Reduced-size Wisp headgear*	1109307
Large Wisp headgear*	1094078
Wisp headgear*	1094082



\*Clips and cushions sold separately

## Replacement parts

Part number	Part number
Wisp headgear clips	1094083
Wisp magnetic headgear clips	1120434



## Replacement parts

Part number	Part number
Wisp clear frame*	1094080
Wisp fabric frame*	1094081



\*Cushion sold separately

# Learn more

For more information about Wisp, visit [www.philips.com/wisp](http://www.philips.com/wisp).





© 2017 Koninklijke Philips N.V. All rights reserved.  
Specifications are subject to change without notice.

[www.philips.com/respironics](http://www.philips.com/respironics)

Caution: US federal law restricts these devices to  
sale by or on the order of a physician.

RRDPGH EB 12/1/17 MCI 4108115 PN 1103630  
1010 Murry Ridge Lane, Murrysville, PA 15668  
800-345-6443 • 724-387-4000