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1. Usage prévu

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1. Intended Use

a. Indications

For directions on using this device, please refer to your Kitett® breast pump instructions.

b. Key performances

Combined with a Kitett® breast pump, the single or double Expression Kit or the KOLOR® breast shield helps

c. Contraindications and adverse effects

A risk of allergy to the materials used (thermoplastic elastomer, dyes) with the part of the body in contact with the breast shield is possible.

2. Product description

a. Single Expression Kit contents for the Kitett® breast pump	b. Double Expression Kit contents for the Kitett® breast pump
1 KOLOR® breast shield	2 KOLOR® breast shields
1 collection container	2 collection containers
1 cap	2 caps/1 interface bottle
1 interface bottle	2 tubing

c. KOLOR® contents

1 KOLOR® breast shield

3. Cleaning and disinfecting

For full-term babies (without medical problems), disinfect before the first use and once a day. For premature babies or those with weakened immune systems, disinfect before the first use and after each use.

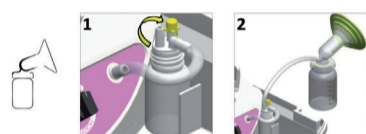
Rinse the parts in contact with the milk in cold water, then clean with hot and soapy water. Rinse all parts. Disinfect all parts in boiling water (at least 10 minutes), in a dishwasher or in a microwave steriliser*, EXCEPT in the case of the single or double Expression Kit, the tubing should briefly be dipped in boiling water.

Recommendation for disinfection with microwave sterilizer: 800w-950w for 4 min.


4. Use

Verify the contents of the single or double Expression Kit or KOLOR® box and the condition of the component parts prior to use.

a. Setting up the single Expression Kit

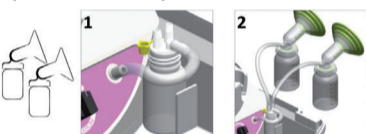b. Setting up the double Expression Kit

c. Setting up the KOLOR® breast shield

For directions on setting up your KOLOR® breast shield, please refer to your Kitett® breast pump instructions.

5. Precautions for use

Do not use with a breast pump with free control. Do not use with a central vacuum hospital system. If the KOLOR® breast shield becomes deformed, immerse it in boiling water for a few minutes.

If signs of degradation or change in colour appear, please replace the accessories.

Do not thaw or heat the collected milk in the microwave.

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